

Instruction Manual



ECG Scanning Software

Software 2.03 and above



SPACELABS
Healthcare

18-0020 Rev. C



© Copyright 2009 Spacelabs Healthcare Ltd

Contents	Page
1 Introduction.....	6
1.1 Disclaimer.....	6
2 Safety and Regulatory.....	7
2.1 Intended Use.....	7
2.2 Declaration of Conformity.....	7
2.3 Caution.....	7
2.4 Warranty.....	8
2.5 Manufacturer.....	9
2.6 Special Conditions.....	9
2.6.1 Installation of Spacelabs Healthcare Software on Third Party PC Systems.....	9
2.6.2 Installation of Other Software or Computer Systems.....	9
2.6.3 Modification of the Operating System.....	9
3 CardioNavigator Operating Instructions.....	11
3.1 Database Operations.....	11
3.1.1 Starting the Program.....	11
3.1.2 Entering a New Patient.....	12
3.1.3 Editing Patient Details.....	13
3.1.3.1 Hiding a Patient in the Database.....	13
3.1.3.2 Restoring a Hidden Patient.....	13
3.1.3.3 Deleting a Patient from the Database.....	13
3.1.3.4 Recovering Deleted Patient Details.....	13
3.1.4 Configuring the Patient List.....	14
3.1.5 Configuring the Patient Details Display.....	14
3.1.6 Sorting and Searching the Patients List.....	14
3.1.7 Filtering the Patient List.....	14
3.1.7.1 Set Filter Definition for the Patient List.....	14
3.1.7.2 Show Only Patients with at Least 1 Recording.....	14
3.1.8 Configuring the Recording List.....	15
3.1.8.1 Selecting the Recording Details.....	15
3.1.8.2 Adding New Columns (Extended Information) To the Recording List 15	15
3.1.9 Filtering the Recording List.....	15
3.1.9.1 Set Filter Definition for the Recording List.....	15
3.1.9.2 Analysed/Confirmed Checkboxes.....	16
3.2 Archiving and Backup.....	16
3.2.1 Creating a New Medium.....	18
3.2.2 Selecting the Recordings to Archive.....	18
3.2.3 Automatically Selecting All Recordings in a Date Range.....	19
3.2.4 Backup for CardioNavigator.....	20
3.3 CardioNavigator Reports.....	20
3.3.1 Retrieving or Reviewing a Stored Report or Recording.....	20
3.3.2 Printing a Stored Report.....	21
3.3.3 Deleting a Stored Report or Recording.....	21
3.4 Data Export Options.....	21
3.5 Email Functions.....	22

3.5.1 Sending or Receiving a Stored Report or Recording by E-mail	22
3.5.2 Marking Items as SENT	22
3.5.3 Reading a Received Email	23
3.6 Starting a Program	23
4 Lifescreen Operating Instructions.....	25
Good Recording Quality	25
4.1.1 Hook-up.....	25
4.1.2 Skin Preparation.....	25
4.1.3 ECG Electrodes.....	25
Lifescreen Instructions.....	26
4.1.4 Main Menu Help	26
4.1.5 Toolbar	26
4.1.6 The Menu Bar.....	26
4.2.4 Tabs.....	26
Loading a Recording and Starting a New Analysis.....	27
4.3.1 For Lifescreen With CardioNavigator Integrated.....	27
4.3.2 For Lifescreen Without CardioNavigator Integrated.....	28
Loading a 7 Day Lifecard CF Recording.....	29
Loading a 48 Hour Lifecard CF Recording.....	29
The Timebar	29
4.7 Patient Details	30
Flowing EGC's.....	31
4.1.7 Controlling the Flowing ECG Display.....	31
4.1.8 Paused Flowing ECG Cursors.....	32
Using Graphs and Histograms	33
4.8.2 RR Tachogram	33
4.8.3 RR Histogram	34
4.8.4 Adjusting the Graph Display	34
4.8.5 Zooming and Re-Scaling	35
Creating and Labelling ECG and Graph Events	36
4.9.1 ECG Strips.....	36
4.9.2 Graph Segments.....	36
4.9.3 General Properties.....	36
4.9.4 Callipers (for ECG Events)	36
4.9.5 Inhibit.....	37
4.10 ECG Annotation Settings.....	37
4.11 ECG Gain and Polarity	38
4.12 Patient Diary.....	39
4.12.1 Add/Edit Patient Diary Entry	39
4.12.2 Importing a Diary List.....	39
4.12.3 Edit Lists	40
4.13 The Print List	40
4.14 Print Report	41
4.15 Saving a Report.....	42
4.16 Managing Reports	44
4.16.1 Export	44
4.16.2 Compact Flashcard.....	45
4.16.3 Compact Flashcard Reader.....	45

4.17 Keyboard Shortcuts	45
4.17.1 The "Active" ECG Channel	45
4.17.2 Shortcut Keys.....	46
4.17.3 Selecting ECG using the keyboard	49
4.18 Demonstration Mode	51
5 Reference	53
5.1 Principles of Operation	53
5.2 Sample Reports.....	54

1 Introduction

You should have the manuals listed below supplied with this product. If any are missing, please contact your supplier immediately for a replacement.

Installation Manual	Part # 47-0049
---------------------	----------------

For Installations with Sentinel	Part # 58-0042
---------------------------------	----------------

Note:

If you are using Lifescreen in conjunction with Sentinel or CardioNavigator Plus please refer the relevant manual below for further details on use.

Sentinel User Manual	Part # 58-0042
----------------------	----------------

CardioNavigator Plus User Manual	Part # 18-0081
----------------------------------	----------------

1.1 Disclaimer

Every effort has been taken to ensure the accuracy of these manuals, but Spacelabs Healthcare cannot accept liability for consequences or losses caused by errors or omissions. You are advised to check with Spacelabs Healthcare on any point on which you are unsure or need confirmation.

2 Safety and Regulatory

2.1 Intended Use

Lifescreeen is an ECG screening system for recording and scanning up to seven days of continuous ECG. The Lifescreeen System consists of the *Lifescreeen* scanning software and the *Lifecard CF 7-Day* digital ECG recorder, pre-configured for Lifescreeen mode.


You may have purchased the *Lifescreeen* software to use in conjunction with a *Pathfinder* ECG analysis system, and your existing Lifecard CF recorders.

Lifescreeen displays the continuous (“full disclosure”) ECG, and displays graphs of average heart rate and individual beat-to-beat intervals, to help you identify interval and rate-related abnormalities. Selected ECG strips and graphs can be printed out in the final report, together with your comments and interpretation. The operator should be trained in electro-cardiography. A qualified medical practitioner should make any diagnosis. Recordings can be exported in a format compatible with a Spacelabs Healthcare *Pathfinder* ECG system, for further detailed analysis.

With *CardioNavigator* you can:

- Enter patient details
- Configure the Lifescreeen or Lifecard CF recorder
- Download recordings from the recorder
- Start a new analysis
- Review a previous analysis
- Print a previous report
- Make a detailed search for specific patients and recordings

2.2 Declaration of Conformity

 This product is in conformity with the essential requirements and provisions of Council Directive 93/42/EEC concerning medical devices. 0120 Your PC equipment should be **CE** Marked to the LV directive and comply with the IT Safety Standard EN60950.

2.3 Caution

In the US, Federal Law restricts this product to use by, or on the order of, a licensed physician.

2.4 Warranty

Subject to the conditions set out below, Spacelabs Healthcare ("The Company") warrants that its Products will be free from defects in material and workmanship for a period of 12 months from delivery. This warranty is given by The Company subject to the following conditions:

- 1 The Company shall be under no liability in respect of any defect arising from fair wear and tear, wilful damage, negligence, abnormal working conditions, failure to follow instructions (whether oral or in writing), misuse, improper installation or alteration or repair of the Products without The Company's approval.
- 2 The above warranty does not extend to parts, materials or equipment not manufactured by The Company, in respect of which the Customer shall only be entitled to the benefit of any such warranty or guarantee as is given by the manufacturer to The Company.
- 3 Subject as expressly provided here, all warranties, conditions or other terms implied by statute or common law are excluded to the fullest extent permitted by law.
- 4 Any claim by the Customer which is based on any defect in material or workmanship of the Products shall be notified to The Company immediately after discovery of the defect. If the Customer does not notify The Company accordingly, the Customer shall not be entitled to reject the Products and The Company shall have no liability for such defect.
- 5 Where any valid claim in respect of any of the Products which is based on any defect in the material or workmanship of the Products is notified to The Company, The Company shall be entitled to replace or repair (at The Company's sole discretion, either at the Customer's premises or at The Company's premises in the United Kingdom) the Products (or part in question) but The Company shall have no further liability to the Customer.
- 6 The Company shall not be liable to the Customer by reason of any representation, or implied warranty, condition or other term, or any duty at common law, or for any consequential loss or damage (whether for loss of profit or otherwise), costs, expenses or other claims for consequential compensation whatsoever arising out of or in connection with any act or omission of The Company relating to the manufacture or supply of the Products or use by the Customer.

2.5 Manufacturer

Spacelabs Healthcare Ltd.
1 Harforde Court
John Tate Road
Hertford, SG13 7NW



Tel: +44 (0) 1992 507700
Fax: +44 (0) 1992 501213

www.spacelabshealthcare.com

2.6 Special Conditions

2.6.1 Installation of Spacelabs Healthcare Software on Third Party PC Systems

The Company accepts no liability for problems introduced in existing systems or software applications, caused by installing this product on third party PC systems (i.e. not supplied by The Company as part of an integrated product). Installation of this product is entirely at the user's own risk. Provision should be made to back up vital data and systems before installing this product.

2.6.2 Installation of Other Software or Computer Systems

This product may fail to operate correctly if you subsequently install other software programs. Obtain authorisation from either your supplier Spacelabs Healthcare before installation. The warranty on this product is INVALIDATED if you proceed with installation without such authorisation. Repairs to systems rendered inoperative as a result may be carried out at the customer's expense and at the Company's discretion.

2.6.3 Modification of the Operating System

This product may fail to operate correctly if you modify the Windows operating system, by installing upgrades, service packs or make changes to the configuration and settings. Obtain authorisation from either your supplier or Spacelabs Healthcare before making any modification. The warranty on this product is INVALIDATED if you proceed with modification without such authorisation. Repairs to systems rendered inoperative as a result may be carried out at the customer's expense and at the Company's discretion.

3 CardioNavigator Operating Instructions

3.1 Database Operations


3.1.1 Starting the Program

NOTE	<i>If you don't have CardioNavigator integrated with your analysis program, you can skip this section and go to Section 4</i>
-------------	--

CardioNavigator may be set up to start automatically on your PC. If not:

- Double click on the *CardioNavigator* Icon on the Windows desktop
- Or
- Click on Start, Programs, CardioNavigator, CardioNavigator

After starting the program, you will see the Patient Database displayed:

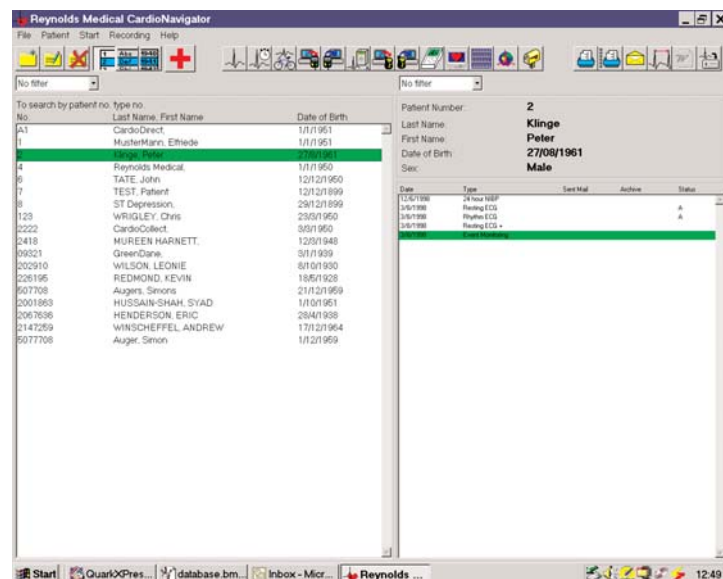

3.1.2 Entering a New Patient

Click on the **New Patient** button

Or

From the menu bar click **Patient** then **New Patient**

Or

Press the **Insert** button on the keyboard

Enter the Patient details as requested in the dialogue box displayed and click on **OK** when done.

Patient Number:	6	Insurance Ref.:	NHS
Last Name:	TATE	N.I. Number:	*****
First Name:	John		
Date of Birth:	12/12/1950		
Sex:	Male		
Weight (kg):	80		
Height (cm):	180		
Post Code:	SG13 7NW		
Town/City:	Hertford		
N°/Street:	1		
County:	Hertfordshire		
Pacemaker:	None		

OK Cancel

NOTE *Fields marked with ! are mandatory. Others are optional*

3.1.3 Editing Patient Details

To change the details for a patient, either double click on their name in the database or click once to highlight their name then click the **EDIT PATIENT** button.

3.1.3.1 Hiding a Patient in the Database

If a patient is not currently undergoing treatment, you can 'hide' their record, without losing any information.

Click once on their name to highlight the entry, then from the menu bar click on **PATIENT** and select **SHOW/HIDE PATIENT**

Check the box to HIDE this patient, then click 'OK'. In a networked setup, this applies to all workstations.



3.1.3.2 Restoring a Hidden Patient

On the menu bar, click on Patient the select **Display Hidden Patients**.

Select the option to **Show Only Hidden Patients**.

In the Patient list, click on the patient you wish to restore, then on the menu bar click on **Patient**, then select **Show/Hide Patient**.





Check **Off** the **Hide** selection then click **OK**.

Finally, change the **Display Hidden Patients** option back to **Show Only Patients Not Hidden**.

3.1.3.3 Deleting a Patient from the Database

Click once to highlight the required entry from the patient list, then click the **Delete Patient** button. (You can also press the **Delete** key on the keyboard or select the menu **Patient - Delete Patient**.)

NOTE	<p> <i>All recordings for this patient will also be deleted.</i></p> <p> <i>You cannot delete a patient if their recordings are archived and the archive medium is unavailable or write-protected.</i></p>
-------------	--

3.1.3.4 Recovering Deleted Patient Details

If you know the Patient Number, you can recover deleted details, however deleted recordings can **ONLY** be recovered from a backup medium.

Select **Recover Deleted Patient Details** from the **File** menu, enter the patient number and click **OK**. The patient record will be restored.

3.1.4 Configuring the Patient List

You can choose which patient details to display in the list, and the order in which they appear, by *right-clicking* in the patient list and selecting the **Configuration** button.

Choose which items you want to be 'visible'. The columns in the patient list will appear in the same order. You may have to use a smaller **Character Size** to display all the information you have chosen.

NOTE	<i>You can add custom fields to the patient's details. See the configuration information in section 4</i>
-------------	--

3.1.5 Configuring the Patient Details Display

Right-click in the Patient Details Display (above the **Recording List**), then click the **Configuration** button. Choose which items you want to be 'visible'. The items will appear in the same order. You may have to use a smaller **Character Size** to display all the information you have chosen. Selecting **Display Hard Disk Capacity** will show the remaining space on the hard disk, and provides a useful warning when space is low and archiving is required.

3.1.6 Sorting and Searching the Patients List

You can sort patients by **Number, Name or Date of Birth**.

Click on the appropriate button to re-order the patient list by **Number, Name or Date of Birth**. If you sort by either number or name, you can *search* by typing (in full) the required number or last name. The highlight bar will move to the corresponding patient record.

3.1.7 Filtering the Patient List

3.1.7.1 Set Filter Definition for the Patient List

Click on the 'Down arrow' beside the Filter field, and then select '**Filter Definition**' to define or setup a new filter for the **Patient List**.

You set up a Filter to list only patient records matching defined criteria. The filter can be very general or very specific- you can use any field appearing in the patient details, including custom fields. If you leave a field blank, it won't be used. For example, you may wish to setup a filter which will group patients together of a certain age and weight.

Click the **New** button and type in a Filter name. Complete the required fields and then click **OK**. Only patients whose details match the filter will be displayed.

3.1.7.2 Show Only Patients with at Least 1 Recording

Checking this box will display only those patients with recordings matching the

RECORDING FILTER. For example, if the recording filter is set for only Lifescreen recordings, only patients with these recordings will be listed.

NOTE	<i>Reselect 'No Filter', from the drop-down list, to display all patients.</i>
-------------	---

3.1.8 Configuring the Recording List

3.1.8.1 Selecting the Recording Details

You can choose which recording details to display, and the order in which they appear, by *right-clicking* in the recording list and selecting the **Configuration** button.

Choose which items you want to be 'visible'. The columns in the recording list will appear in the same order. Depending on your monitor setup, you may need to change the order and the Character size, to make the columns visible.

Status Field

A = **A**nalysed (12-Lead rhythm ECG).

B = **B**ackground 12-Lead rhythm stored.

C = **C**onfirmed 12-Lead Report.

T = **exT**ended information set.

3.1.8.2 Adding New Columns (Extended Information) To the Recording List

User-defined fields can be displayed in additional columns in the Recording Details. You must define these fields first in the Options (see the Installation Manual), after which they can be added to the 'Visible' list, above. For example: to add a list of Recorder Types. To assign a value from your list to an individual highlighted recording, select **Extended Recording Information** from the **Recording** menu.

3.1.9 Filtering the Recording List

3.1.9.1 Set Filter Definition for the Recording List

Click on the 'Down arrow' beside the Filter field, and then select '**Filter definition**' to define or setup a new filter for the **Recording List**.

You set up a Filter to list only recordings matching defined criteria. The filter can be very general or very specific- you can use any field appearing in the recording details, including custom fields. If you leave a field blank, it won't be used. A check-mark means this type of recording *will* be listed; no check-mark means it *won't* be listed. For example, you may wish to setup a filter which will display only a certain type of recording, over a certain date range.



Click the **New** button and type in a Filter name.

Complete the required fields and then click OK. When you highlight a Patient Record, only recordings whose details match the filter will be listed.

3.1.9.2 Analysed/Confirmed Checkboxes

'Use' = Filter inspects this setting

'State' = Analysed/Confirmed if checked, Not Analysed/Confirmed if not checked.

NOTE	<p>The 'Analysed' and 'Confirmed' status can only be used with 12-Lead ECG recordings from CardioNavigator/CardioCollect</p> <p> To Limit the Patient List, see 'Show Only Patients With At Least 1 Recording.'</p> <p> Reselect 'No Filter', to display all recordings.</p>
-------------	---


3.2 Archiving and Backup

Your CardioNavigator database will gradually become full as the hard disk space is used up.

The **Cardioarchive** program is part of *CardioNavigator*, and is used to move recordings from their 'default' location (normally the hard disk) to an '**Archive Medium**', such as a DVD RAM, recordable CD or a network drive. This frees up space on the hard disk and enables you to transfer the database to a more secure medium.

Note that the recordings are *moved*. After archiving the database, there is still only one copy, on the new medium. The patient details are *copied*. That is, the patient's name and details still appear in the *CardioNavigator* database, so that you can add new recordings to that patient. To start the archive program, click on '**Start Archive Program** in the CardioNavigator **File** menu:

The CardioArchive program and database will be displayed:

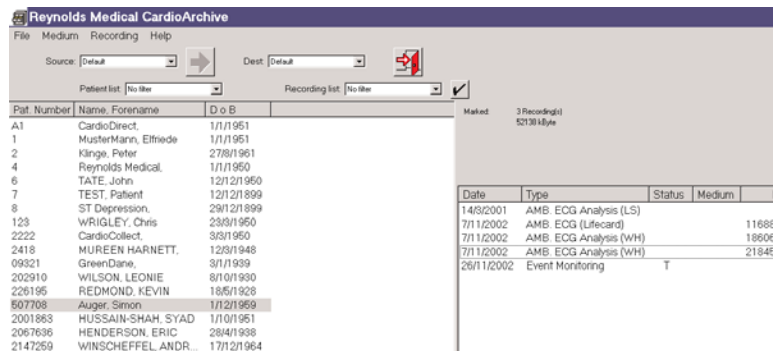

The **Source** specifies *from* where you want to move recordings. 'Default' is normally hard disk 'C' on the PC.

The **Dest** (destination) specifies *to* where you want to move recordings. Sources and Destinations must appear as drive letters ('logical drives') to the PC, for example 'F:', 'H:', 'M:'. CardioArchive currently does not support UNC names (for example \\server\archive\).

If you want to use a network drive, it must be mapped as a drive to the PC. Please ask your IT department for help with network drives.

Select the source or destination archive by clicking on the down arrow adjacent to '**Source**' or '**Dest**', then click on the required name.

CardioArchive keeps a 'master' list of patient names and recordings, for all archives. When **All Archives** is selected as the **Source**, CardioArchive lists all patients and the name of the archive where their recordings are located.

3.2.1 Creating a New Medium

A new medium must be electronically named before you can use it for archiving. If you are using a writable CD or DVD, it must be formatted and/or made writable by your CD/DVD writer software, before attempting to name it with CardioArchive.

When the medium is ready, select **Medium – Create New Medium** from the menu:

In the next box, select the correct drive letter from the list of **Available Drives** and enter a name to identify the medium. For example, 'January -March 2003':

Click **OK** to complete the operation. This name will now appear in the list of Source and Destination drives. **Be sure to write the name of the archive on the CD/DVD label!**

NOTE	<i>A CD or DVD can only have one archive. You cannot create 'January - March 2003' and 'April -June 2003' on the same CD, for example. A new CD is required for the second archive. In the case of network drives, a new folder must be created and shared on the network.</i>
-------------	---

3.2.2 Selecting the Recordings to Archive

The **left**-hand window lists all the patients with recordings in the Source archive. If a patient has no recordings, they won't be listed here (because there is nothing to archive). The **right**-hand window lists all recordings on that archive, for the selected (highlighted) patient.

6	IATE, John	12/12/1960
7	TEST, Patient	12/12/1999
8	ST Depression,	29/12/1999
123	WRIGHTLEY, Chris	23/01/1950
2222	CardioCollect,	3/07/1950
2418	MUREEN HARNETT,	12/01/1948
09321	GreenDane,	3/1/1939
202910	WILSON, LEONIE	8/10/1930
226195	REDMOND, KEVIN	18/07/1928
507708	Auger, Simon	1/12/1959

Date	Type	Status	Medium	M
14/02/2001	AMB ECG Analysis (LS)			
7/11/2002	AMB ECG (Lifecard)		11688K	
7/11/2002	AMB ECG Analysis (WH)		18609K	
7/11/2002	AMB ECG Analysis (WH)		21846K	
26/11/2002	Event Monitoring	T		

You can select individual patients and recordings by clicking on the patient's name, then double-clicking on the recordings you want to archive. For example, here all recordings for November 2002 have been selected for patient number 507708:

When you double-click on a recording, the size of the recording appears in the column **'M'** (medium). If you want to deselect a recording, double-click on it again.

3.2.3 Automatically Selecting All Recordings in a Date Range

Most often you will probably want to archive all recordings in a certain date range. You do this by first setting up a **Recording Filter**, which specifies the date range.

Click on the down-arrow adjacent to '**Recording List**', and select **Filter Definition**

Click on **New** to open a new filter template, and enter a name (for example, 'January March 2003'). In the new template, type in the appropriate **Date Range**.

NOTE	<i>The 'Analysed' and 'Confirmed' status can only be used with 12-Lead ECG recordings from CardioDirect/CardioCollect. Leave this blank for all other types.</i>
-------------	--

Filter Definition for Recording List [X]

January-March 2003 [v] New Rename Delete

Resting ECG
 Resting ECG +
 Rhythm ECG
 Stress Test
 24 hour NIBP
 Event Monitoring
 AMB. ECG (Lifecard)
 AMB. ECG Report (PA)
 AMB. ECG Report (PA-p)
 AMB. ECG Analysis (PA)
 AMB. ECG Report (LS)
 AMB. ECG Analysis (LS)
 AMB. ECG Report (wH)
 AMB. ECG Analysis (wH)

Date Range 1/1/2003 . 31/3/2003

Analysed use state
Confirmed use state

OK Cancel

On the left, check all the recording types you want to archive. In this example, we selected all recording types: Click **OK** to apply this filter.

In the database, you will now see only patients and their recordings, matching the filter criteria. If a patient has recordings from other dates, these recordings won't be listed. If a patient has no recordings that match, their name won't appear in the left-hand list.

To quickly select all patients and recordings in this display, click the **'Tick'** button on the screen then click **OK** to accept the **'Select Marked'** option.

Choosing **Release Marked** de-selects all previously marked recordings.

Reverse Marked changes marked to unmarked, and vice-versa.

Make sure the correct Destination archive medium is selected, before clicking the **'Right Arrow'** button to start archiving the recordings.

When archiving is complete, this message advises you to write-protect the medium, to prevent accidental overwriting or erasure.

NOTE	<i>If you are using a CD-R, this is called 'making compatible', after which the CD-R cannot be written to again.</i>
-------------	---

When you return to the CardioNavigator database, if a selected patient has archived recordings, the archive is listed in the Archive column of the **Recording List**. The Archive name is the one you gave when creating that new archive medium. If you want to view those recordings, the corresponding archive medium (CD, DVD, or network drive) must be ready.

3.2.4 Backup for CardioNavigator

The Backup utility makes a copy of the database to a DVD RAM, recordable CD or a network drive. This can be used to recover data if the PC hard disk is damaged. The Backup utility must first be set up in the Options (see the Installation Manual). You may want help from your IT/IS department with this. Once the utility has been set up, you can make a backup at any time by selecting Backup for CardioNavigator from the File menu:

When you run Backup for CardioNavigator, the database must close first- click OK when you are ready.

3.3 CardioNavigator Reports

3.3.1 Retrieving or Reviewing a Stored Report or Recording

Click on the required patient in the **Patient List** whose recording you want to retrieve. Then *double-click* on the recording in the **Recording List** to retrieve or review it. It will be opened with the appropriate program.

3.3.2 Printing a Stored Report

To print a stored report directly, without opening it, first highlight the required report in the **Recording List**, then click on one of the 'printer' buttons on the toolbar:




Print report in your 'Standard Format', as defined in the Options (see the Installation Manual).



Choose the format for printing the report. (This is only available for 12Lead ECG, Ambulatory BP and Event Recorder reports).

3.3.3 Deleting a Stored Report or Recording

Highlight the required report or recording in the Recording List.


Click the  button. (You can select more than one report or recording for

the same patient by holding the **Control** key and clicking, or the **Shift** key to select a range.)

NOTE	<p><i>Once you confirm deletion, the reports or recordings cannot be recovered.</i></p> <p><i>You cannot delete archived items unless the archive medium is ready and is write-enabled.</i></p>
-------------	---

3.4 Data Export Options

These options must be set up first in the CardioNavigator Options (see section 4.22). The option button will become active on the toolbar when the appropriate report type is highlighted in the **Recording List**, (otherwise it is 'greyed-out').

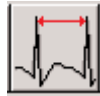

Export QRS Intervals from 12-Lead Rhythm ECG and Event Recordings only. The data format is described in Section 6.



Export Exercise Stress ECG Report in Word/RTF format. A sample is shown in Section 6.



Export RAW ECG Data from 12-Lead Rhythm ECG, Resting ECG and Event Recordings only. The data format is described in Section 6.




Holter Text File Saves, in text format, the patient demographic details and analysis results summary from a Pathfinder ambulatory ECG report. The data format is described in Section 6.

3.5 Email Functions

3.5.1 Sending or Receiving a Stored Report or Recording by E-mail

Sending a report by email is an additional option available for CardioNavigator. It requires a dongle (protection key). You can send or receive all reports and recordings *except* for Lifecard CF raw ECG (“AMB. ECG (Lifecard)”). The recipient must also have the CardioNavigator program. The system must have an existing email account. The email option must first be set up in the Options (see the Installation Manual). You may want help from your IT/IS department with this.

- 1 Highlight the required report or recording in the **Recording List**, then click the  **Send Mail** button
- 2 Select the recipient from the address list then click **OK**
- 3 The report or recording will be attached to an email form. Add any further comments, then send the email as usual.

3.5.2 Marking Items as SENT

In the Options, set up an ‘Extended Information’ item for the Recording List, (See the Installation Manual for details), and configure the recording list to display this field (see page 15). You could label the field ‘Sent Mail’, with options ‘Yes’ and ‘No’, for example.

When highlighting the item to send by email, select **Recording – Extended Recording Information** and choose ‘Yes’.

3.5.3 Reading a Received Email

Open the received email message as usual. (Your email program may be set up to alert you when new mail arrives.)

Double-click on the 'attachment' and (depending on the email program you are using), *Open*, *Run* or *Launch* it. (Don't select the Save option). The recording will be copied to the Received Mail List - it will not be opened immediately.


Click on CardioNavigator's  **Received Mail** button, and then select the required item from the list.

Click **Assign** to compare this recording with the database.

NOTE	<i>A matching patient record is NOT selected automatically</i>
-------------	---

The **!** mark shows where the patient data does not match the currently highlighted patient in the database.

If you know this is a new patient, click **New Patient** to generate a new patient record, then click **OK** to assign the recording.

If the patient record exists (the **!** mark would appear if some details were omitted or mistyped), highlight the correct existing record and assign the recording. The **'Find'**  buttons snap to the matching entry in the database, if it exists.

 button on the

NOTE	<i>Only click 'OK' when you are sure the correct patient record is highlighted.</i>
-------------	--

A message similar to these may appear when you close the original email message. This is normal, because the attachment has been moved. You don't need to save the changes and no data will be lost.

3.6 Starting a Program

All your Spacelabs Healthcare analysis programs can be started from the central toolbar, or from the **File** menu.



The appropriate program will also be opened when you double-click a report, analysis or recording in the **Recording List**.

Start a Lifes

4 Lifescreen Operating Instructions

Good Recording Quality

Good recording quality is your first priority

4.1.1 Hook-up

Proper hook-up, or patient connection, is essential to ensure intelligible ECG recordings. Inadequate skin preparation will result in excessive artefact in the signal, causing measurement of false intervals and difficulty in visual interpretation.

4.1.2 Skin Preparation

There are 3 main steps in good skin preparation:

CLEAN the skin using isopropyl alcohol, soapy water or a non-alcoholic swab. This removes grease, which prevents the electrode adhering and impairs conductivity.

SHAVE the site as needed with a new disposable razor - the electrode will not remain in place on hair!

RUB the site with dry gauze to lightly redden the skin. This removes dry, dead skin, lowering impedance and reducing pick-up of interference.

NOTE	<i>Take care not to break the skin or cause bleeding. Avoid abrasion (rubbing) on patients with delicate skin, including children and elderly patients.</i>
-------------	--

4.1.3 ECG Electrodes

Always use good quality electrodes designed for long-term (Holter) recording, and ensure adequate skin preparation. We recommend Ambu Blue Sensor VL-00-S for 24 hour recordings, and VLC-00-S for extended recordings up to 7 days. Further information can be found at www.ambu.com.

Apply the electrodes in the positions shown in the ECG recorder instruction manual.

Check the ECG quality on the recorder, as shown in the recorder instruction manual.

Lifescreeen Instructions

These instructions can also be found in the Lifescreeen Online Help. To read the Help, start the Lifescreeen program and click on Help – Help topics:

4.1.4 Main Menu Help

To display this, you click **Help** and then **Help Topics** from the main menu. The Main Menu Help is a series of pages which explain the features of Lifescreeen. When you see some blue or purple text, underlined (like this), it is called a hyperlink. When you place the cursor over this, the cursor changes to a hand

You can then click on the text, to display more help on the subject.

You can also click on help topics on the left side of the Help Window, to display the corresponding page.

Help Within a Window or ‘Dialog Box’

When you are in one of the menu windows or ‘dialog boxes’, click on the question mark button at the top right of the dialogue, the cursor changes to a question mark. You can then click on any of the items, for example buttons or edit boxes, and a short help message will be displayed. If you click outside the items, the cursor will return to normal.

4.1.5 Toolbar



The **Toolbar** is the row of pictorial ‘icons’ or ‘buttons’, under the Menu Bar at the top of the display. Clicking on one of the buttons is the same as choosing the relevant option from one of the menus. If you put the mouse cursor over a toolbar button without clicking, a ‘tool tip’ appears, telling you what that button will do.

4.1.6 The Menu Bar

The **Menu Bar** is the row of words, or menu headings, at the top of the display. Clicking on one of the headings displays a drop-down menu of options. Several menu options also have a Toolbar button that performs the same function.

4.2.4 Tabs

When the Lifescreeen display is shown in ‘split screen’ (ECG above, graphs below), you can change which graph or table is shown in the bottom part of the screen by clicking the appropriate tab.

You can switch to split screen mode by clicking the **Split** Screen button on the toolbar. You can switch to full-screen ECG by clicking the **Full-Screen** button.

You can arrange the tabs to be along either the top or the bottom of the display, by selecting **Arrange Tabs** in the **View** menu option. Also see how to adjust the graph display.

Loading a Recording and Starting a New Analysis

4.3.1 For Lifescreen With CardioNavigator Integrated...

1 Select the Patient



Select the patient in the CardioNavigator database (or enter new patient details by clicking the **New Patient** button).

2 Option 1: Store the Recording, Analyse Later



Insert the compact flashcard recording into the compact flashcard reader, connected to your PC. Now click the **Lifecard CF - Download** button on the toolbar, to store the recording in the **Recording List**. You may then re-use the card for another patient.

To analyse the recording, double-click the stored recording in the list

NOTE	<i>Routinely delete stored recordings you don't need, to save disk space</i>
-------------	---

2 Option 2: Load the Recording and Analyse Now



Insert the compact flashcard recording into the compact flashcard reader, connected to your PC. Now click the **Lifescreen** button on the toolbar. The recorded ECG will be loaded and displayed. If you have a previous analysis open and not saved it, you'll be prompted to do so, as all results of the previous analysis will be erased

4.3.2 For Lifescreen Without CardioNavigator Integrated...

1 Option 1: Store the Recording, Analyse Later


- From the Lifescreen menu, select **File – Manage Reports**. The Report Manager will start up.
- From the Report Manager menu, select **Recording – Lifecard CF – Store in Archive**.
- Fill in the required patient details, and then click **OK**.

NOTE	<i>Unless you intend to email this recording, ensure Compress Recording is NOT selected.</i>
-------------	---

- To analyse the recording, double-click the stored recording in the list.

NOTE	<i>Routinely delete stored recordings you don't need, to save disk space, and empty the DELETED RECORDINGS Archive (selected from the Archive drop-down list).</i>
-------------	---

2 Option 1: Load the Recording, Analyse Now

- Insert the compact flashcard recording into the compact flashcard reader, connected to your PC. Now from the menu, click on File and select Load Recording.
- You can also click on the **Load Recording**  button, on the toolbar. The recorded ECG will be loaded and displayed. If you have a previous analysis open and have not saved it, you will be prompted to do so, as all results of the previous analysis will be erased.

Loading a 7 Day Lifecard CF Recording

When analysing a 7-Day recording, you can choose to load all days, or make a selection. However you cannot have 'gaps' in the selection.

Click the button '**on**' for the consecutive days you wish to load and analyse. To view any remaining days that you switched '**off**', you will need to load the recording again and make a new selection. If you load all the days now, you can still restrict the analysis to one day, by using the Timebar, for example. At the top of this display, there is a button to **Play Sound Recording**, which will replay the patient's name or ID, if it was recorded by the operator when the recorder was fitted.

Loading a 48 Hour Lifecard CF Recording

If you are loading a 'standard' Lifecard CF recording (up to 48 hours), you can choose to either analyse the first 24 hours or the remainder of the recording. Each part of the recording must be analysed and reported separately. After completing the report on the first 24 hours, you should load the remainder of the recording. A separate report is produced for each part, they are not combined. Recordings made in this mode cannot be loaded in one pass, because of the large amount of high-definition data they contain. (See also 7-Day Recordings). There is a button to **Play Sound Recording**, which will replay the patient's name or ID, if it was recorded by the operator when the recorder was fitted.

The Timebar



The timebar, at the top of the ECG display, shows the duration of the loaded and analysed recording. The timebar is annotated with (Day) hours:minutes. You can immediately jump to any time in the recording by left-clicking on the timebar.

The **yellow** cursor indicates the current time.

The **green** area of the timebar indicates the analysed period.

The **brown** area of the timebar indicates the loaded, but unanalysed period.

The **grey** area of the timebar indicates how much ECG is still to load. You can restrict the analysis (that is, the period for which the ECG is displayed and the graphs calculated) to one day or a selected range. If you left-click (or right-click), drag and highlight part of the timebar, you can select from the pop-up menu:

- Analyse selected area (the area you highlighted in **blue** on the timebar)
- Analyse the entire recording
- Analyse the first day
- Analyse a specified day. Choose the day from the sub-menu.

From the same menu you can inhibit or un-inhibit the beats in that region. This may be useful if one part of the recording contains a lot of noise or artefact.

You can Export the ECG in the selected **blue** area, for further analysis using a Spacelabs Healthcare Pathfinder analyser.

4.7 Patient Details

The **Patient Details** are displayed by selecting Patient Details from the **View** menu, or by clicking the **Patient Details** button on the toolbar.

The Patient Details will be printed on the front page of the report. You can type in details to identify the **Patient**, their referring **Doctor** and **Hospital**, and the Lifescreen system **Operator**.

At the bottom of this dialog, there is a button to **Play Sound Recording**, which will replay the patient's name or ID, if it was recorded by the operator when the recorder was fitted.

Some details are completed automatically, by reading details from the Lifecard recording (for example, **Date of Recording**, **Recorder Number** and **ID**. (You can read more information on these in the Lifecard CF user manual.) The **Analysis Date** is entered automatically from the PC system clock.


There are 3 'tabs' for comments, and each has a drop-down list of comments you might frequently use:-

- **Symptoms or Reason for Test**, where you could note the reasons for the original referral, or the symptoms noted by the patient during the recording.
- **Operator's Comments**, which may be a summary of what was seen in the recording, to assist the doctor with their interpretation.
- **Doctor's Comments**, where the doctor can enter a diagnosis and recommendations.

See also the Edit Lists option and Patient Diary Entries.

Flowing EGC's

The flowing ECG display 'writes' ECG on the screen like a conventional ECG machine. The ECG is written from left to right, and when one page is full, it begins overwriting from the top.

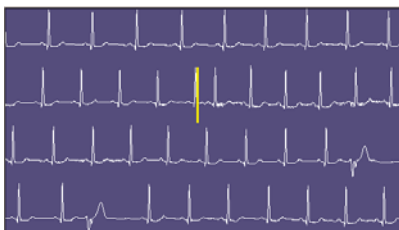

Start by clicking the **Flowing ECG** button

With this selected, you can adjust the speed by dragging the 'Speed Control' on the toolbar, left or right, or by pressing the **'arrow keys'** on the keyboard.

To stop, just click anywhere on the ECG display, or press any key on the keyboard.

You can display 1,2 or 3 channels of ECG by making a selection in the ECG Display Settings menu. You can also click the **'cal pulse'** symbols on the left of the display, or press 1, 2 and 3 on the keyboard to turn each channel on or off.

4.1.7 Controlling the Flowing ECG Display

- **If Flowing ECG is *NOT* active:**
Pressing the space bar (or clicking the run button on the toolbar) will start Flowing ECG.
- **If Flowing ECG *IS* active**

Pressing the space bar *or* the 'Pause' key will pause and resume Flowing ECG.

Right-clicking will pause and resume Flowing ECG exactly like the space bar does.

Double-clicking the ECG will return you to the normal display, with the ECG cursor centred at the location that was double-clicked. This will happen whether or not the flowing ECG is paused.

Single clicking the ECG will pause the display (if it isn't already paused) and move the "point of interest" cursor to the clicked area. If Flowing ECG is already paused, it just moves the "point of interest" cursor. (See below for more information about the Flowing ECG cursors.)

Pressing the Escape key or the Enter key will stop Flowing ECG, and return you to the normal display with the cursor centred about the “point of interest”, as defined above. You can also click the “stop” button on the toolbar.

The following keys will increase the flowing speed:

- ‘+’, **right-arrow**, **up-arrow** (on either keypad, with or without numlock)
- Rotating the mouse wheel towards you will also increase the run speed.

The following keys will decrease the flowing speed:

- ‘-’, **left-arrow**, **down-arrow** (on either keypad, with or without numlock)
- Rotating the mouse wheel away from you will also decrease the run speed.

4.1.8 Paused Flowing ECG Cursors

When Flowing ECG is paused, there will be *two* vertical bar cursors displayed.

The green cursor shows the “rewrite point”, i.e. where the next portion of ECG is about to be displayed. This represents a discontinuity - the ECG in front of the green cursor is later in time than the ECG after it.

The yellow cursor shows the “point of interest” that has been clicked on by you. You can click to reposition the point of interest as many times as you like while Flowing ECG is paused.


If the two cursors are in the same place (which they always will be if you press the spacebar to pause the display), only the yellow one will be displayed.

Using Graphs and Histograms


To display a graph or histogram, click on the tab at the bottom left of the display, or select from the **View** option on the menu bar.

Lifescreeen can display graphs for:

4.8.1 Heart Rate Graph - the rate in beats per minute, averaged over every whole minute of recording time. All beats that are identified are included. (See also 'Trigger Arrows' in the Annotations section).

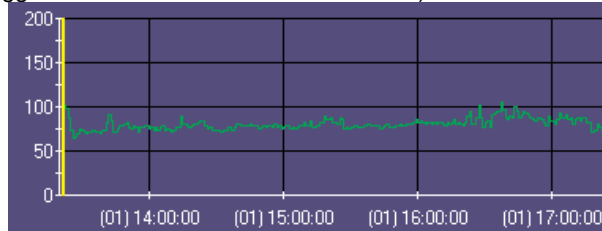

4.8.2 RR Tachogram -this displays every detected RR (beat-to-beat) interval. Adjacent points are joined by lines, except where there is an inhibited region in the recording.

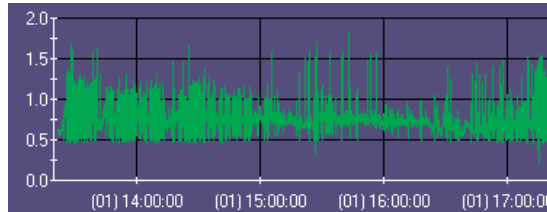


Possible ectopic intervals are readily observed as a big 'stick' in the graph. Click on the 'stick' to see the corresponding ECG.

You can inhibit intervals from the analysis using the tachogram. Left-click, drag and highlight the region of interest- a yellow box indicates the area- and from the pop-up menu select **Inhibit**. You can reverse this action by reselecting the area and choosing **Un-inhibit**. Inhibited areas are shown as a 'gap' in the graph

Undo and Redo

You can undo or redo the last inhibit action you took, by selecting the appropriate option from the **Edit** menu.

4.8.3 RR Histogram –is a ‘bar chart’ recording the number of RR intervals within a range. The ‘width’ of each bar in the histogram is 50 milliseconds.

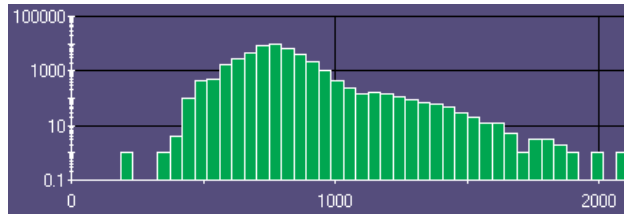


The histogram is drawn with a logarithmic vertical scale, as standard. This makes it easier to see low and high values on the same axis. You can change to a linear scale, by clicking the **Log log** button at the top right of the graph.


When you click on a ‘bar’ in the histogram, the ECG display shows you:


- the longest RR interval in the ‘bar’ if you are to the right of the median
- the shortest RR interval in the ‘bar’, if you are to the left of the median

The rightmost bar contains all intervals of 5 seconds or longer.

The interval range and the number of intervals in the ‘bar’, and the current interval, are displayed at the top of the histogram.



To see the next longest interval, click the ‘**Right Arrow**’  button, or press the ^ up or > right keyboard arrow. To jump to the next bar, hold down the **Control** key on the keyboard at the same time.

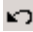To see the next shortest RR, click the **Left Arrow**  button, or press the v down or <-left keyboard arrow. To jump to the next bar, hold down the **Control** key on the keyboard at the same time.

To go back to the last RR you viewed, click the **Back**  button or press **Control** and **Z** on the keyboard.



By selecting a region of the histogram, you can restrict the analysis to a certain day, by selecting **Analyse** from the pop-up menu. The graph results are restricted to only the day you specify. You can subsequently return the selection to the Entire Recording.

4.8.4 Adjusting the Graph Display


The standard display is a ‘split screen’, with ECG at the top, and a graph at the bottom. You can switch between split screen and full screen ECG selecting Full Screen ECG from the View option in the menu, or by clicking the **Full Screen** ECG  button or the **Split Screen**  button on the toolbar.

You can re-size the graph by pointing the cursor at the grey 'border' between the ECG. When the cursor shape changes to a double-headed arrow, click and drag the border up or down.

4.8.5 Zooming and Re-Scaling

Click on the  or  buttons at the top of the graph to zoom in or out of the graph, or use the + or – buttons on the keyboard. The graph zooms around the yellow cursor line.

To zoom or re-scale a graph, left-click, drag and highlight the region of interest - a yellow box indicates the area. From the pop-up menu select Zoom, then choose which axes to zoom.

To return a graph to full scale, click the **Left/Right**  arrow button at the top of the graph, or select **Re-Scale/Unzoom** from the pop-up menu.


Also see the Print List.

Creating and Labelling ECG and Graph Events

You can select areas of the ECG and the graphs for inclusion in the printed report. These selected areas are called 'Events' or 'Event Strips'

4.9.1 ECG Strips

Left-click and drag, to select a region of the ECG, then choose **Create ECG Strip** from the pop-up menu. You can also do this by pressing the Print Strip button


You can set up 'properties' for the event from the window that is displayed. 


4.9.2 Graph Segments


Left-click and drag, to select a region of the Heart Rate Graph or RR Tachogram, then choose **Create Graph Segment** from the pop-up menu. You can set up 'properties' for the event from the window that is displayed. All your events will be shown in the Print List, where you can re-edit them.

4.9.3 General Properties

The **Start Time** and **Duration** of the selected region are shown at the top of this display window.

The **Event** name is the title you want to give to this event. You can choose from the list of names for ECG strips, by clicking the  down-arrow and selecting from the list. This list can be edited using the **Edit Event Names List** function. The name for Graph Segments is fixed.

Comments can be appended to the name, by clicking the  down-arrow and selecting from the list. This list can also be edited using the **Edit Event Names List** function.

Choose the appropriate **Print Scale** by clicking the  down-arrow and selecting from the list. Larger scales use more paper. For example, if you have selected 30 seconds of ECG, it will require 1 page of paper if printed at 25mm/s (8 seconds per strip). The scale for Graph Segments is fixed.

4.9.4 Callipers (for ECG Events)

Click the check-box to include the calliper **Time Measurement** on the strip. This is the duration that was highlighted in the selection. You can also add the **Beat-Beat Rate**, which converts the time measurement to a heart rate. The calliper doesn't detect the beats within the highlighted region, so take care to highlight precisely from one beat to the next. For display zoom sizes x1, x2 and x4, you can click the check-box to include the **Amplitude Measurement**, indicated by the yellow box in the selected area.

The amplitude is in millimetres and is automatically calibrated (1mV ECG = 2 large squares on the printed grid). This is generally of significance only for ST segment deviation measurement. Calliper measurements are not applicable to Graph Segments.

To Undo and Redo

You can undo or redo the last action you took on an event, by selecting the appropriate option from the Edit menu.

4.9.5 Inhibit

Inhibited areas are parts of the recording where the noise/artefact level is too high. Lifescreen will display the recording, but will exclude it from the analysis. The beats and intervals will not be used in the graphs, and the beats will have no annotation.


You can inhibit additional areas manually, if they cause erroneous results, by highlighting the area in the ECG, or on the timebar, and selecting **Inhibit**.

Manually inhibited areas can also be **uninhibited**, in the same way.

To Undo and Redo

You can undo or redo the last inhibit action you took, by selecting the appropriate option from the **Edit** menu.

4.10 ECG Annotation Settings

Selecting **ECG Annotation Settings** from the Options menu, displays the ECG Annotations Settings dialog box. You can also display this by clicking the button on the toolbar. 

There are several options you can select under the **Global Settings** tab: **Grey out Inhibited ECG** colours grey any areas where you have highlighted and selected Inhibit Beats, or where Lifescreen has detected an excessive noise level. It is useful to see how much of the recording is inhibited. **Display Pacing Markers** draws a yellow line in the ECG, wherever the recorder detected a pacing pulse in the surface ECG. Reliable detection of the pacing pulse requires good hook-up. Sometimes noise creates a false pacing marker. **Annotate Inhibited Beats** will place a greyed out arrow below all beats that you have manually inhibited. **Disable All Annotation** leaves the ECG with no annotation at all. You can save any changes permanently by checking the **Save Settings** box. By clicking the Zoom tab for each individual **Zoom** range, there are several options you can select:

Display Trigger Arrows places an arrow beneath every detected beat. Sometimes noise may be detected as a beat, or a poorly projected or 'noisy' beat may not be detected at all. The trigger arrows are very useful alerting you to these problems when you are reviewing beat intervals.

Display Event Markers 'underlines' all areas of ECG which you have marked as an event. This is useful when reviewing your analysis.


Display RR Intervals displays the interval in milliseconds between consecutive detected beats. The standard configuration does not display this at smaller zoom ranges, to avoid overcrowding the display.

Display Channel Delineation Lines draws a line under each ECG row. You may find this makes viewing easier.

To change the size on screen of the **Polarity Icon** ('cal pulse' indicating normal or inverted ECG), or the recording **Time** annotation, click the **Icon/Font** tab, then choose the setting you require.

You can save any changes permanently by checking the **Save Settings** box.

4.11 ECG Gain and Polarity


Selecting **ECG Display Settings** from the **Options** menu will display the ECG Display Settings dialog box. You can also display this dialog box by pressing the **Display Settings**  button on the toolbar.

Here you can make adjustments to the way ECG is displayed, and you can make individual changes for each ECG channel by selecting the appropriate 'tab'. You can save your new settings permanently by checking the **Save Settings** box. You can adjust the Gain for each channel, in steps from x1/4 to x4. These changes do not affect the detection of beats, but have a big effect on the clarity of the ECG on the screen. You may want to use normal or lower gain when first reviewing the ECG, so that more can be displayed, then increase the gain in areas of interest, to reveal any important detail in the ECG. The **Vertical ECG Offset** control moves the ECG up or down. Use this adjustment to bring a region of ECG back into view, if it is affected by baseline wander. (You would not usually save this setting). Too much baseline wander indicates poor hook-up and skin preparation! Check the **Invert Polarity** box to invert the ECG in that channel.

Under the **Settings** tab, you can change the **Zoom** size of the display, in steps from x1/8 to x4. You can also zoom the display by pressing the + and – keys on the keyboard, or by double-clicking in the ECG. Increase the Zoom setting to magnify a region of interest, or reduce it to display more ECG.

The **ECG Display** settings select which channels of ECG appear on the screen. Normally all 3 channels are on view, but you can switch off any channel which is unusable, or perhaps have only one channel displayed for maximum clarity. You can also switch channels on/off by clicking the square 'Cal Pulse' symbols on the left side of the display, or by pressing the 1, 2 and 3 keys on the keyboard (each press turns that channel on/off).

The **ECG Row Separation** control adjusts the amount of space between rows of ECG. Use this setting in conjunction with the Gain and Zoom controls, to prevent ECG overlapping between rows.

Click the **Centre Cursor**  button to centre the currently highlighted ECG in the display.

4.12 Patient Diary

If you asked the patient to keep a written diary, Lifescreen can automatically create an ECG event strip at the specified times. Select the Enter Patient Diary option from the File menu. To enter a diary time, click the Add button.

4.12.1 Add/Edit Patient Diary Entry

To edit any entry you have made to the Patient Diary list, highlight the required entry and click the **Edit** button. By using the drop-down lists, you can select a different date, time, symptom or activity. You can also simply type over any item (except for the date) If the changes do not already exist in the list, you will be asked if you want to save them for future use. If you answer 'No', they'll be saved only for that diary entry. If you decide you don't want one of the entries before accepting the list, highlight it then click the **Delete** button. Finally, click OK to accept the list.

An event strip will appear in the Print List and will be printed in the report. The diary entries will be removed from the **Enter Patient Diary** list. The Print List shows how many times in each day that the patient pressed the '**Patient Event**' button on the recorder, to mark the onset of symptoms. This may help to identify the most significant days in the recording. Lifescreen automatically creates an ECG Event Strip at these times, which you can view from the Print List. The digital recorder records the ECG continuously, not only when the event button is pressed.

4.12.2 Importing a Diary List

You could arrange for an administrative assistant to prepare the diary list first, by typing details into a text document, using a word processing program or Windows **Notepad**. The details must be typed in the format <DATE><space><TIME><space><COMMENT>, using a new line for each diary entry. For example:

07/04/2001 19:30:00 Chest pain while climbing stairs.

07/04/2001 20:10:00 Palpitations.

07/04/2001 9:15PM Breathless

The time can be entered in either 24 hour format or 12 hour (AM/PM), as shown. You don't have to enter the number of seconds. The date format automatically accepts your local setting (European or North American). Activities and symptoms cannot be entered separately in this list, which must be saved in a text format (type '.txt'). Click on the **Import** button and locate the text file, to import the list. A message will alert you if there is any mistake in the file format

4.12.3 Edit Lists

By selecting **Edit Lists** from the **Options** menu, you can customise the drop-down lists available to label event strips and the comments in the report. Click one of the 5 'tabs' to select a list.

Event Names are the 'titles' of ECG events.

Event Comments may be appended to the name you choose for the event.

Operator Comments and **Doctor Comments** are added to the front page of the printed report, in the Patient Details.

Activities are added to information you enter in the Patient Diary. In each case, you can:

Add, add a new item to the list. Another box will appear where you can type in your chosen text.

Edit, modify an existing item. Any item can be edited.

Delete, remove this item permanently from the list.

To return **ALL** the lists to the original factory settings (losing all changes), click the **Reset to System Default** button.

4.13 The Print List

All the ECG strips and graph segments you have selected for printing are listed in the **Print List**. To view the list, click on the **Print List** tab at the bottom of the display.

	Event	Start Time	Duration	Print Scale
1	Couplet	(1) 14:26:26		25mm/sec Strip
2	Heart Rate Graph Segment	(1) 15:59:07	00:52:06	25mm/sec Strip
3	PVC	(1) 17:53:48		25mm/sec Strip
4	Patient Event	(1) 17:54:01		25mm/sec Strip
5	Patient Event	(1) 17:55:05		25mm/sec Strip
6	Patient Event	(1) 17:57:22		25mm/sec Strip
7	Salvo	(1) 20:34:13		25mm/sec Strip

When you left-click on an event on the list, it will be shown on the ECG display. The display will jump to that time.

To edit an event, double-click on the event in the list. It will be shown in the ECG display and the Print Properties will appear.

When you right-click on an event in the list, it will be shown on the ECG display, and from the pop-up menu you can either **Edit** the event or **Delete** the event. If you select **Edit**, you can change the details of the event from the Print Properties.

You can delete multiple events by selecting them together in the list. To select consecutive events, left-click on the first one, and hold the button down whilst dragging down to select the others. Release the button, then right-click on the highlighted region to display the pop-up menu. To select non-consecutive events, hold down **Control** on the keyboard, while you left-click in turn on the required events. Then, right-click on the highlighted region to display the pop-up menu.

Undo and Redo

You can undo or redo the last action you took on an event, by selecting the appropriate option from the Edit menu.

4.14 Print Report

Selecting **Print or Preview Report** from the File menu provides several choices. Click the **Preview** button to see how the report will look before printing out. When you are ready to print out, click the **Print** button, or click **Cancel** if you do not want to print now.

Report Format

You can change the Report Format by clicking the down-arrow from the list. Two standard report formats are available:

- **Full Report**, with patient details, all graphs and ECG strips,
- **Short Report**, with patient details and ECG strips only.

You can create and customise your own report format, using the **Set Up Report Formats** option in the **File** menu. If you want the format you selected to be the default (so it is pre-selected in the list), check the **Set as Default** box.

Printer

If you have more than one printer set up to use with this PC, you can choose one from the list after clicking the down-arrow


Print Range and Copies


You can choose to print all pages or only a selected range of pages, by entering the page numbers in the box. You can also increase the number of copies that you want to print.

Set Up Report Formats

Basics - By selecting **Set Up Report Formats** from the **File** menu, you can customise the appearance of printed reports. You can modify an existing format and either save it under the same name (if it was previously customised), or under a new name. You cannot save changes to the standard System Formats - they are protected. But you can create a new format based on a system format, then use a new name when saving.

Procedure

Begin by selecting the Report you want to modify, by clicking the down-arrow  and highlighting the name in the list. The **Report** column describes what you want to print. The **Layout** column describes where you want to print it. You can choose the **Page Frame** (the border around the page) and the appearance of the **Header** (title on the page), and define up to 8 **Sections** for the report. The sections do not need to start on a new page.

Click the down arrow  to the right of each item, to make a selection from the list.

After you have made all your selections, click **OK**. Another box will appear with the following options:

Change This Format, modifying the original format and saving it with the same name, or

Create New Format, saving it with a new name and preserving the original format. You must choose this option if you started by modifying a system format

Other Functions

You can delete a customised format by selecting it from the list, then clicking the **Delete** button. You cannot delete system formats.

You can change the name of a customised format by clicking the **Rename Report Format** button, typing a new description in the box that then appears. You cannot change the name of a system format.

4.15 Saving a Report

There are two types of report that you can save, Full Reports or Formatted Reports. Full Reports contain all the ECG and analysis data, and can be reloaded to continue your analysis. Formatted Reports are a copy of a printed report, which can be viewed and emailed using Report Manager

To save a Full Report, select **Save Full Report** from the **File** menu. To save a Formatted Report select **Save Formatted Report** from the **File** menu. These selections display a dialog that allows you to select the destination of the report save operation.

The **Report Name** is taken automatically from the patient's name that you entered in the Patient Details dialog, but you can replace this with any name you choose.

The **Location** specifies the disk and folder where the report is to be saved. If there is more than one location set up on your system, you can choose the one you want by clicking the down-arrow and selecting from the list. You can add more locations by using the **Configure Folder** menu option in the **Report Manager** program - see the user manual for more details.

Store Report To defines the 'archive' where this report will be stored, within the selected location. If there is more than one 'archive' set up on your system, you can choose the one you want by clicking the down-arrow and selecting from the list. You can add more archives by using either the **Configure Folder** or **Add Archive to Configuration** menu options in the **Report Manager** program.

If you are saving a Formatted Report, **Report Format** allows you to select which format the saved report should have.

You can also Export all or part of the recording, to a Del Mar Reynolds Medical Pathfinder analyser.

Confirm Overwrite Report

Lifescreeen gives you the option either to replace an original report or create another new report.

After loading a Lifecard ECG recording from a flashcard:-

When you proceed to store a report more than once during the current analysis, Lifescreeen gives you the option to replace the original or create another new report. You might choose to replace the original if you had made a correction to it and the original was invalid.

You might choose to make a new copy if you need to maintain a history or audit trail of changes made to the report. If you re-load the same recording, or load a new recording, you will not see this option when you first store the report, even if the report has the same name. This is because Lifescreeen knows it is from a different analysis 'session'.

After loading a previously stored full report:

Lifescreeen always gives you the option either to replace the original or create another new report.

Recover an Autosaved Analysis

Lifescreeen can recover from a power failure or PC 'crash', if you have been using the Autosave feature (this is normally enabled).

When you next start Lifescreeen normally, the autosaved analysis will be retrieved. This includes all your event strips, comments and patient details, plus any beat inhibits you had made, up to the time of the last autosave.

If you had been analysing ECG from a flashcard recording when the problem occurred, you will need to reinsert the card into the flashcard reader. The ID will be checked and the original ECG reloaded, before applying the autosaved analysis. If you were analysing a stored recording from an archive, it will be retrieved and the autosaved analysis applied. To help avoid losing your work in the event of a power failure or failure in your PC, Lifescreeen can automatically save the current analysis. You can set the frequency of automatic saves from 5 – 120 minutes. If you don't want to use this feature, clear the check-box.

4.16 Managing Reports

You can search, copy, move, delete, rename and reorganise all your stored reports using the Report Manager program.

Click on **Manage Reports** in the Lifescreeen **File** menu, to start the Report Manager.

Loading a Report When you select **Load Report** from the **File** menu, the **Report Manager** program is opened. This allows you to view all the reports you have previously saved, and to select a Report to load into Lifescreeen.

NOTE	<i>You can only load Lifescreeen Reports.</i>
-------------	--

4.16.1 Export

You can export all or part of the recording in a special format, for further analysis using a Del Mar Reynolds Medical Pathfinder analyser. To export the entire recording, select Export to Pathfinder from the File option on the menu bar. To export part of the recording, highlight the required region on the timebar, then select the Export Selected Region option from the pop-up menu.

4.16.2 Compact Flashcard

The compact flashcard (CF Card) is the recording medium for the Lifecard CF recorder. The card is 36 x 42 mm, and is available with several memory capacities. In normal use, the data stored on the card is very secure. It will not be erased unless it is reprogrammed, or physically abused. Your Lifecard recorder user manual gives details of the amount of ECG that can be recorded on a given capacity card, using a particular patient cable. If you need additional CF cards, you should buy them from Del Mar Reynolds Medical, to ensure compatibility with your Lifecard recorder and Lifescreen system.

4.16.3 Compact Flashcard Reader

The compact flashcard (CF Card) reader is connected to your PC via the USB (Universal Serial Bus) socket, and is used to read ECG and patient information from the CF card, as recorded by the Lifecard CF recorder. Make sure you insert the card correctly, with the Spacelabs Healthcare logo facing down and the lined label facing up. Never try to force the card, as the reader may be damaged.

4.17 Keyboard Shortcuts

Advanced users may find Lifescreen's keyboard shortcuts to be useful.

Each of the main display windows – the graphs and the ECG window - has its own set of keyboard shortcuts. These shortcuts will only work when the window has the "input focus". You give the input focus to a window by clicking within it. You can tell if a window has the input focus because the window's cursor will be flashing when the window has the focus. So, for example, the ECG cursor will be flashing while the ECG window has the focus, and the graph cursor will be flashing when the graph window has the focus.

4.17.1 The "Active" ECG Channel

Some of the keys affect the "active" ECG channel. Only one channel may be "active" at any one time. The "active" status for a channel is only used when you press a keyboard shortcut – you can ignore the "active" status at all other times

To select a channel as the "active" channel, click on its polarity icon on the left of the ECG window. The active channel always has its polarity icon drawn in yellow. If a channel is not displayed, its polarity icon is drawn in grey. If a channel is displayed, but not the "active" channel, its polarity icon is drawn in green.

4.17.2 Shortcut Keys

Global Shortcut Keys

These keys can be used when any main window has the focus. Note that they will have no effect while a dialog box is displayed:

Shift Keys	Keystroke	Effect
Ctrl	F	Toggle full-screen ECG display
Ctrl	L	Toggle log axis of histogram, if histogram is visible
Alt	R	Toggle display of RR intervals in the ECG display
Alt	A	Displays the ECG Annotation Settings dialog
Alt	S	Displays the ECH Display Settings dialog
Alt	P	Invert active channel (toggle polarity)
Alt	T	Toggles display of ECG times in ECG window
Alt	F10	Print strip at current ECG cursor location
	F3	Load new recording
	F5	Display Patient Details dialog
Alt	+	Zoom in ECG display
Alt	-	Zoom out ECG display
	Tab	Switch focus between ECG and Graphs/Print Lists

Basic ECG Keyboard Shortcuts

Shift Keys	Keystroke	Effect
	Up Arrow	Move ECG cursor up
	Down Arrow	Move ECG cursor down
	Left Arrow	Move ECG cursor left (wrapping at left of window)
	Right Arrow	Move ECG cursor right (wrapping at right of window)
Ctrl	Home	Move ECG cursor to start of ECG
Ctrl	End	Move ECG cursor to end of ECG
	Page up	Move ECG cursor up one page
	Page Down	Move ECG cursor down one page
	Home	Move ECG cursor to the left of the current row
	End	Move ECG cursor to the right of the current row
Ctrl	Left Arrow	Move ECG cursor left by a large amount
Ctrl	Right Arrow	Move ECG cursor right by a large amount
Alt	Left Arrow	Move ECG cursor left by a small amount
Alt	Right Arrow	Move ECG cursor right by a small amount
	Insert	Slide ECG left
	Delete	Slide ECG right
Ctrl	Insert	Slide ECG left by a large amount
Ctrl	Delete	Slide ECG right by a large amount
Alt	Insert	Slide ECG left by a small amount
Alt	Delete	Slide ECG right by a small amount
	<Spacebar>	Toggle running display mode
	+	Zoom in ECG
	-	Zoom out ECG
	5 (on keypad)	Centre ECG cursor in display

Advanced ECG Short

Shift Keys	Keystroke	Effect
Shift	A	Toggles display of ECG Annotation
Shift	F	Changes the size of the time text (on the left of the windows)
Alt	G	Changes the size of the polarity icons (on the left of the window)
Shift	H	Toggles grey-outs for inhibited ECG
Alt	I	Toggles display of the polarity icons (on the left of the window)
Alt	M	Toggles display of event markers
Ctrl	P	Toggles display of pacing markers
Ctrl	Up Arrow	Increase ECG row separation by a large amount
Ctrl	Down Arrow	Decrease ECG row separation by a large amount
Alt	Up Arrow	Increase ECG row separation by a small amount
Alt	Down Arrow	Decrease ECG row separation by a small amount
Ctrl	+	Increase gain of active ECG channel
Ctrl	-	Decrease gain of active ECG channel
Ctrl	Page Up	Shift active ECG channel up by a large amount
Ctrl	Page Down	Shift active ECG channel down by a large amount
Alt	Page Up	Shift active ECG channel up by a small amount
Alt	Page Down	Shift active ECG channel down by a small amount
Alt	1,2 or 3	Activate channel 1,2 or 3, if it is currently displayed
Shift	1,2 or 3	Display only channel 1,2 or 3
	1,2 or 3	Toggle display of channel 1,2 or 3

4.17.3 Selecting ECG using the keyboard

It is possible to select ECG using the keyboard instead of the mouse. To do this, first move the ECG cursor to the start or end of the area that you want to select. Then press and hold either <shift> key while you move the ECG cursor by pressing the ECG cursor movement keys (up, down, left, right, page-up, page-down, home, end and so on). When you have selected the desired area of ECG, release the <shift> key to display the ECG area selection menu.

Tachogram and Heart Rate Graph Keyboard Shortcuts

These keys can only be used while the tachogram or heart rate graph has the

Shift Keys	Keystroke	Effect
	+	Zoom in
	-	Zoom out
	* (keypad)	Display entire graph
	Home	Move cursor to left of visible portion
	End	Move cursor to right of visible portion
Ctrl	Home	Move cursor to start of graph
Ctrl	End	Move cursor to end of graph
	Page Up	Scroll graph to left
	Page Down	Scroll graph to right
	Left Arrow	Move cursor one second to the left
	Right Arrow	Move cursor one second to the right
Shift	Left Arrow	Move cursor one minute to the left
Shift	Right Arrow	Move cursor one minute to the right
Ctrl	Left Arrow	Move cursor one hour to the left
Ctrl	Right Arrow	Move cursor one hour to the right
Ctrl+Shift	Left Arrow	Move cursor one day to the left
Ctrl+Shift	Right Arrow	Move cursor one day to the right
Alt	Left Arrow	Move cursor to the left ¼ display width
Alt	Right Arrow	Move cursor to the right ¼ display width

Histogram Keyboard Shortcuts

Shift Keys	Keystroke	Effect
	+	Zoom in
	-	Zoom out
	* (keypad)	Display entire graph
	Home	Move cursor to left of visible portion
	End	Move cursor to right of visible portion
Ctrl	Home	Move cursor to start of graph
Ctrl	End	Move cursor to end of graph
	Page Up	Scroll graph to the left
	Page Down	Scroll graph to the right
	Left Arrow	Show previous beat in ECG display
	Right Arrow	Show next beat in ECG display
Ctrl or Shift	Left Arrow	Move cursor to previous bar
Ctrl or Shift	Right Arrow	Move cursor to next bar
Ctrl	z	Undo last cursor move. (Can undo many moves)
Alt	Left Arrow	Move cursor to the left $\frac{1}{4}$ display width
Alt	Right Arrow	Move cursor to the right $\frac{1}{4}$ display width

“Print List” Keyboard Shortcuts

These can only be used while the “Print List” window has the focus. The “Print List” doesn’t have a cursor that flashes when it has the focus. Instead, the current row will be highlighted in bright white when it has the –focus and in grey when it does not.

Shift Keys	Keystroke	Effect
	Delete	Delete the currently highlighted strip
	Return	Edit the currently highlighted strip
	Up Arrow	Select previous strip
	Down Arrow	Select next strip
	Page Up	Move up one page
	Page Down	Move down one page
Ctrl	Home	Go to first strip in the list
Ctrl	End	Go to last strip in the list

4.18 Demonstration Mode

If you have not purchased Lifescreen, you can use it in Demonstration Mode. In Demo Mode you can only open sample reports from the Lifescreen CD-ROM. You cannot load new recordings. When you purchase Lifescreen, you will receive a 'dongle' (security key) and a license number, so that you can make full use of the program.

Opening a Sample Report

Put the Lifescreen CD-ROM into your CD/DVD drive. Wait for the installation program to begin, but then click **Cancel**, as you do not need to proceed with the installation.

If you have Lifescreen Integrated with CardioNavigator...

From the Lifescreen **File** menu, select **CardioNavigator**. This switches to CardioNavigator. Now select **FILE-REPORT MANAGER**

If Lifescreen is NOT Integrated with CardioNavigator...

From the **File** menu, select **Open Report**.

In either case, you will now see the **Report Manager**.

In the Report Manager, from the **File** menu, select **Find Archive Folders**. Click on your CD/DVD drive:


(This should have the Lifescreen icon), then click **OK**.

The Demo Report Archive will be opened. Double-click on a report, to open it in Lifescreen.

The '**Formatted**' report is a stored copy of a printed report, which will be reviewed using the Report Manager.

The '**Lifescreen**' full report is a complete, analysed, edited report with ECG strips, and will be reviewed using Lifescreen.

The sample '**Lifecard CF Recordings**' are original ECG recordings that will be loaded into Lifescreen.

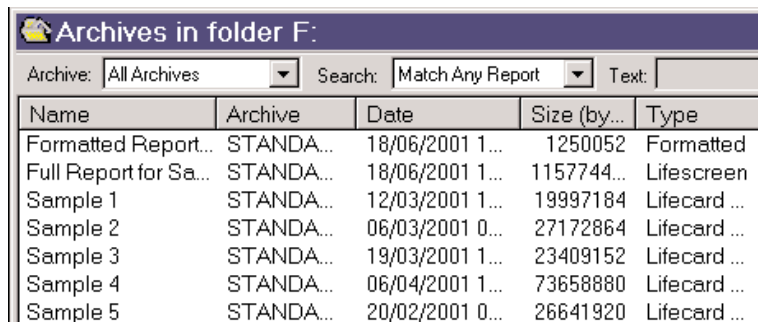


Name	Archive	Date	Size (by...	Type
Formatted Report...	STANDA...	18/06/2001 1...	1250052	Formatted
Full Report for Sa...	STANDA...	18/06/2001 1...	1157744...	Lifescreen
Sample 1	STANDA...	12/03/2001 1...	19997184	Lifecard ...
Sample 2	STANDA...	06/03/2001 0...	27172864	Lifecard ...
Sample 3	STANDA...	19/03/2001 1...	23409152	Lifecard ...
Sample 4	STANDA...	06/04/2001 1...	73658880	Lifecard ...
Sample 5	STANDA...	20/02/2001 0...	26641920	Lifecard ...

**Please
Note:**

When you load a recording from a CD-ROM, click OK on the warning message that says “The recording cannot be marked as read”. An original recording on a compact flashcard is electronically marked to show it has been read, but this cannot be done to a recording stored on a CD-ROM.

Some functions (for example, changing patient details or loading new data) cannot be performed in demonstration mode. You will see a message like this

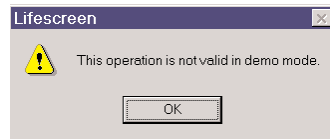

5 Reference

5.1 Principles of Operation

Lifescreeen has a sophisticated beat detection ("trigger") software algorithm, based on the Pathfinder ambulatory ECG analyser. There are no adjustments in Lifescreeen, making it simple to operate. Lifescreeen triggers on the best quality channel in each minute of the recording. The measured beat-to-beat intervals are used to create the heart rate graph, RR histogram and RR tachogram. Lifescreeen displays ECG sampled at 128Hz (7.8ms resolution). The amplitude resolution is 2.5 μ V, with 12-bit (4096) digital resolution.

5.2 Sample Reports

This is a sample of a "Full Report":

Reynolds Medical Lifescreen				
Ambulatory ECG Report				
Patient Name : Reynolds Medical			Patient No. : 4	
Printed Strip List				
Number	Event	Start	Duration	Comments
1	Isolated VBI	(1) 20:56:15		
2	Pause	(4) 03:09:16		
3	SVT	(4) 13:09:53		

 Page 2 of 4

Reynolds Medical Lifescreen
Ambulatory ECG Report

Patient No.	: 4	Recording Date	: 14/03/2001
Patient Name	: Reynolds Medical	Recording Duration	: (5) 00:18:57
Date of Birth	: 01/01/1950	Analysis Date	: 04/03/2003
Sex	: Male	Operator	:
Patient NI Number	:	Physicinn	:
Insurer	:	Hospital / Practice	:
Recorder Number	: 00002005	Analysr	: Lifescreen Version 2.01

Symptoms or reason for test :

Patient reported palpitations.

Operator's Comments :

Occasional ventricular extrasystoles. Episodes of supraventricular tachycardia. Pauses.


Doctor's Comments :


The patient is to begin medication.


Operator's Signature Date


Doctor's Signature Date



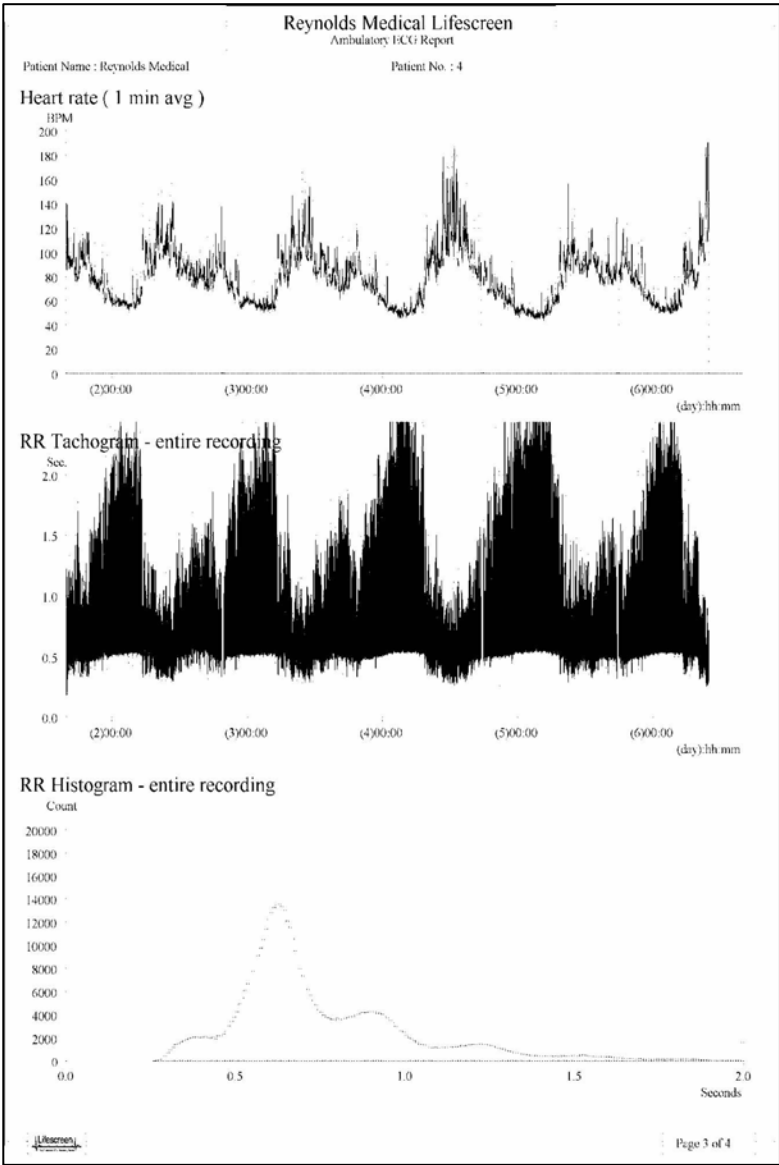

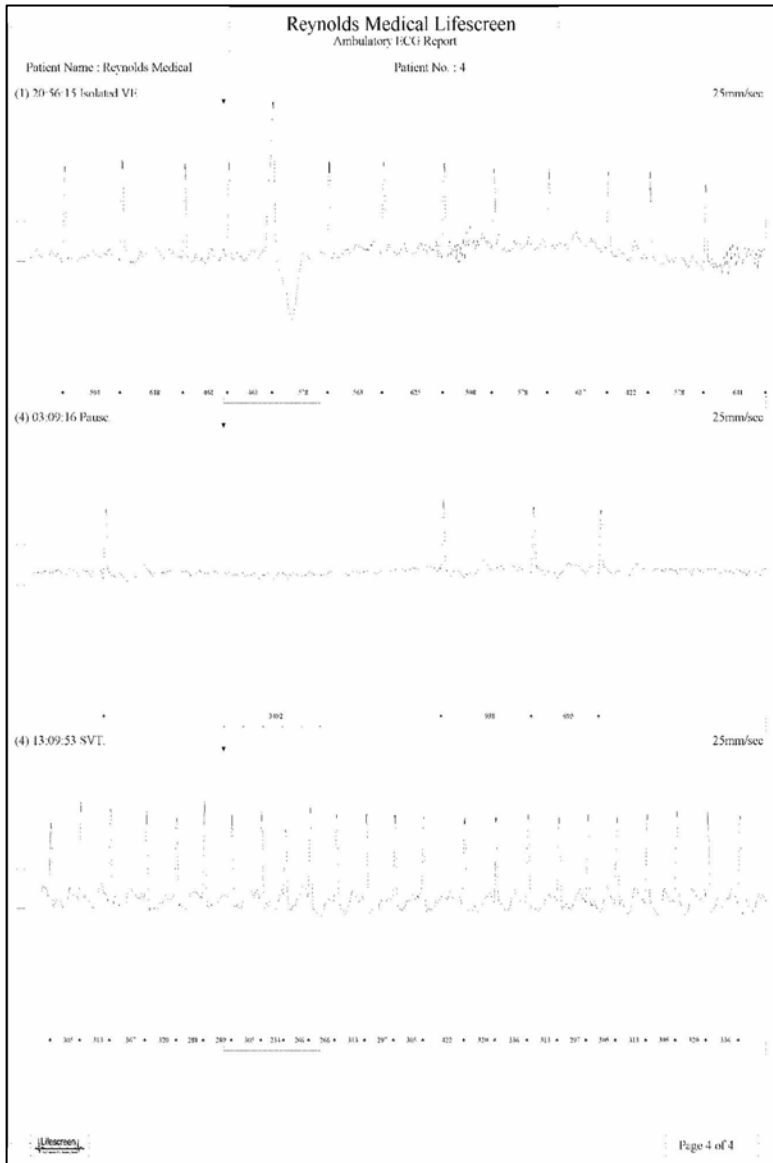

Reynolds Medical Lifescreen			
Ambulatory ECG Report			
Patient No.	: 4	Recording Date	: 14/03/2001
Patient Name	: Reynolds Medical	Recording Duration	: (5) 00:18:57
Date of Birth	: 01/01/1950	Analysis Date	: 04/03/2003
Sex	: Male	Operator	:
Patient NI Number	:	Physician	:
Insurer	:	Hospital / Practice	:
Recorder Number	: 00002005	Analysr	: Lifescreen Version 2.01
Symptoms or reason for test :			
Patient reported palpitations.			
Operator's Comments :			
Occasional ventricular extrasystoles. Episodes of supraventricular tachycardia. Pauses.			
Doctor's Comments :			
The patient is to begin medication.			
		Operator's Signature	Date
		Doctor's Signature	Date
		Page 1 of 2	

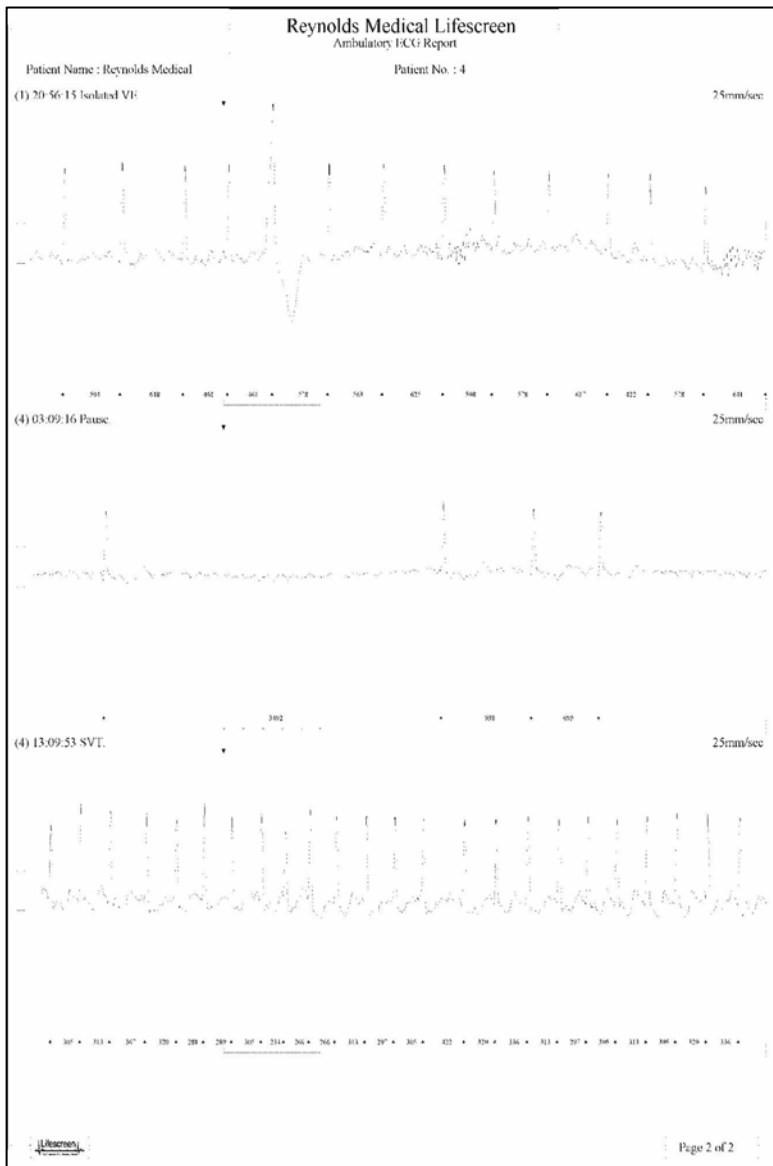



Serving The HealthCare Industry



1800 67 5432



0800 300 421



1800 67 5433



0800 300 422



sales@ecomед.com.au



sales@ecomед.co.nz



www.ecomed.com.au



www.ecomed.co.nz

www.spacelabshealthcare.com

Spacelabs Healthcare inc.
5150 220th Ave. SE
Issaquah
WA 98029, USA

Tel: +1 425 657 7200
Fax: +1 425 657 7212

Spacelabs Healthcare Ltd.
1 Harforde Court
John Tate Road
Hertford, SG13 7NW, UK

Tel: +44 (0) 1992 507700
Fax: +44 (0) 1992 501213

R_x only

