

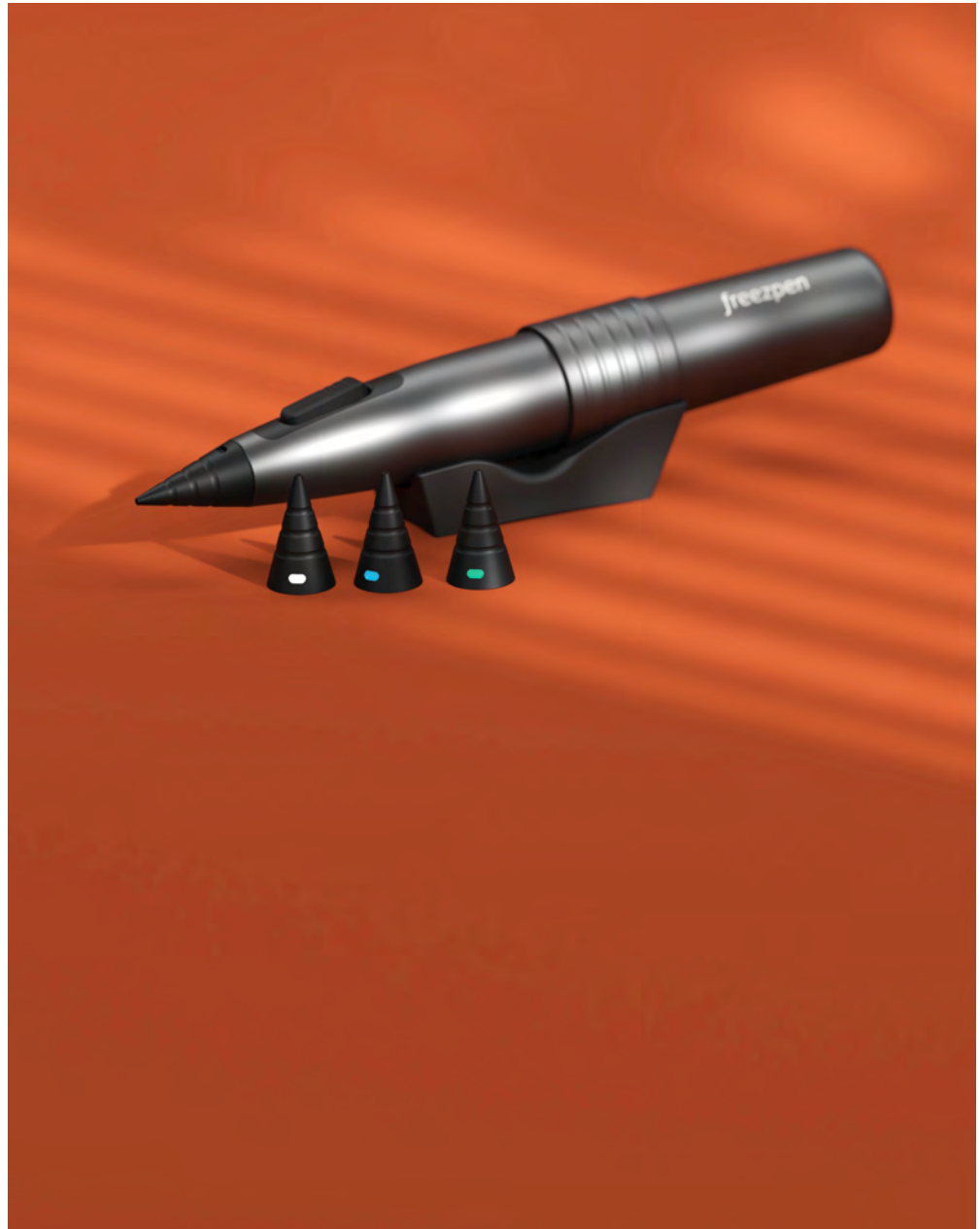


## FREEZPEN 16G



Freezpen is the ideal solution for the cryogenic treatment of benign skin lesions – it's powerful, precise and practical, delivering a great patient experience and a great return on investment.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

As this model works with our 16g cartridges, it's ideal for regular use and treatment of larger or deeper lesions.



### Key features

- Ergonomic design for superior control
- Leading-edge microtechnology ensures controlled gas flow
- Takes easy-to-load 16g N<sub>2</sub>O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 3 applicators
- Suitable for regular use
- Certified CE1639  





## APPLICATORS



Ø 1-3mm  
Power: ■



Ø 2-6mm  
Power: ■■



Ø 4-8mm  
Power: ■■■■

### Continuous flow time with a 16g cartridge



## PRODUCT INFORMATION

### Tutorial

Scan or click:



### Device

The Freezpen's ergonomic and familiar pen-like design ensures the ultimate control for safe, precision treatment.

### Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

### What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, granuloma annulare, molluscum contagiosum and seborrheic keratosis.



## CONTENTS OF THE STARTER SET

### Device

Freezpen 16g

### Applicators

1-3mm applicator  
2-6mm applicator  
4-8mm applicator

### Cartridges

6x 16g cartridges

### Accessories

Hard carrying case  
Forceps  
Device stand

### Documentation

Treatment overview  
User manual

## BASICS

### Dimensions

Device: 20.3 x 3.1 x 3.1 cm  
Carrying case: 27 x 22.9 x 6.4 cm  
Starter set: 25.5 x 24.5 x 11.3 cm

### HS codes

90189084 (device)  
28112930 (cartridges)

### Weight

Device with a full cartridge: 280g  
Starter set: 1,702g

