



## FREEZPEN 35G


Freezpen is the ideal solution for the cryogenic treatment of benign skin lesions – it's powerful, precise and practical, delivering a great patient experience and a great return on investment.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

As this model works with our largest 35g cartridges, it delivers more power and is suitable for intensive use and treatment of larger or deeper lesions.


### Key features

- Ergonomic design for superior control
- Leading-edge microtechnology ensures controlled gas flow
- Long cartridge life with easy-to-load 35g N<sub>2</sub>O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 3 applicators
- Suitable for intensive and varied use
- Certified CE1639 FDA MDSAP

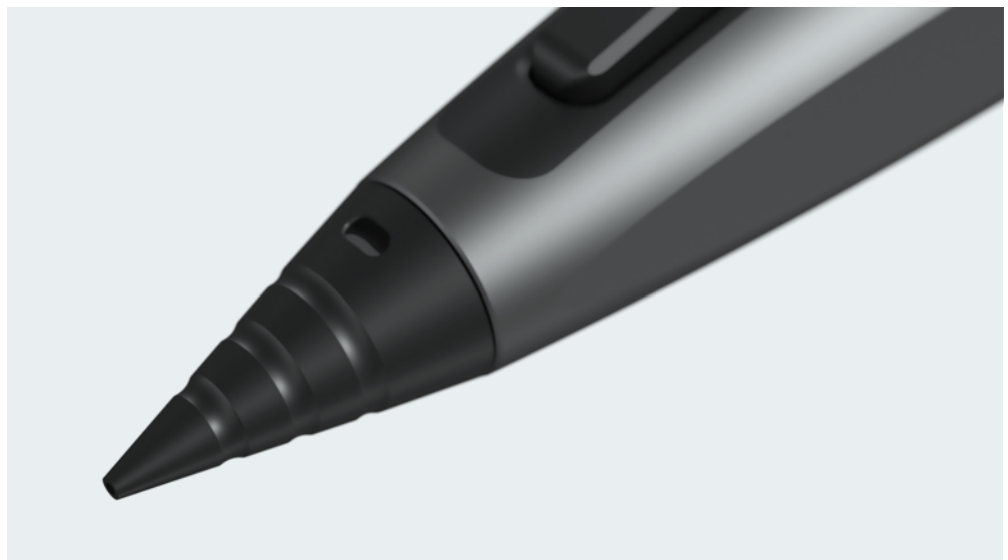



## APPLICATORS

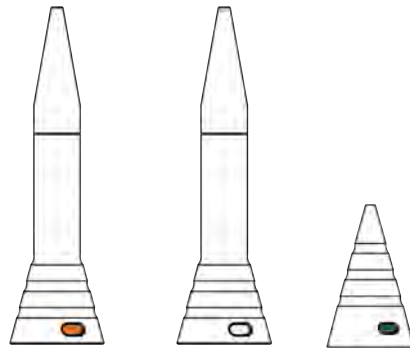

Ø 1-3mm

Power: ■


Ø 2-6mm

Power: ■ ■

Ø 4-8mm

Power: ■ ■ ■ ■

### Continuous flow time with a 35g cartridge

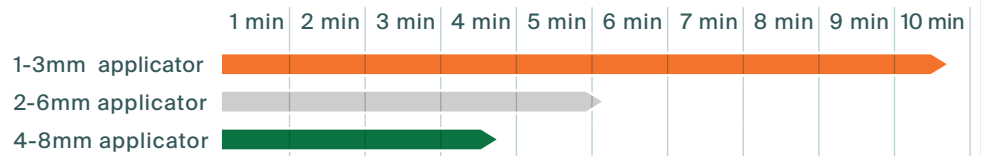

## PRODUCT INFORMATION

### Tutorial

Scan or click:



### Device

The Freezpen's ergonomic and familiar pen-like design ensures the ultimate control for safe, precision treatment.

### Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

### What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, granuloma annulare, molluscum contagiosum and seborrheic keratosis.



## CONTENTS OF THE STARTER SET

### Device

Freezpen 35g

### Applicators

1-3mm applicator long  
2-6mm applicator long  
4-8mm applicator

### Cartridges

4x 35g cartridges

### Accessories

Hard carrying case  
Forceps  
Device stand

### Documentation

Treatment overview  
User manual

## BASICS

### Dimensions

Device: 21.8 x 3.4 x 3.4 cm  
Carrying case: 27 x 22.9 x 6.4 cm  
Starter set: 25.5 x 24.5 x 11.3 cm

### HS codes

90189084 (device)  
28112930 (cartridges)

### Weight

Device with a full cartridge: 320g  
Starter set: 1,931g

