

HOW TO USE Quick Manual





1. Turn on the System

Press the power button(\circlearrowleft) until the device turns on.

2. Select Patient Type



3. Apply Ultrasound Gel

Apply gel on the top of the probe before scanning.

* Patient Position: Have the patient lie in the supine position





HOW TO USE

Quick Manual

4. Aim Toward the Bladder

Put the gel-applied probe on the patient's abdomen and aim toward the bladder.

Important: Please make sure that the Scan button of the probe should be facing the foot side of the patient.





5. Locate the Bladder by Pre-scan Function

Press the Scan button to start Pre-Scan, and then the Pre-Scan function shows the live bladder image.

Important: Please make sure that the Pre-Scan function is 'on' in the device set-up.



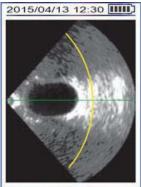

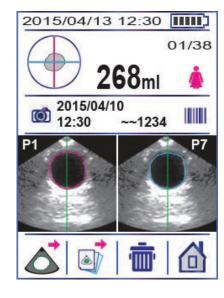

6. Scan Result Screen

When the bladder image looks the biggest on the screen, press the Scan button.

Important: When the 'Pre-Scan' function is 'off' in the device set-up, a scanning will be done without Pre-Scan stage.

7. Save and / or Print



1800 67 5432

1800 67 5433

⊠sales@ecomed.com.au

www.ecomed.com.au

2 0800 300 421

₫ 0800 300 422

⊠sales@ecomed.co.nz

www.ecomed.co.nz