

Vitalograph®



asma-1 Child



**Guide for Healthcare
Professionals**

Testing Subject

- 1 Insert mouthpiece and ① switch on



- 2 Blow 3 times



- 3 ▼ View values for parameters FEV₅, FEV₇₅, FEV₁ and PEF



4 Best blow



PEF +/- 5%; FEV₁ +/- 3%

EN ISO 23747:2009

Specifies accuracy: $\pm 10\%$ PEF

Resistance to flow: < 0.35 kPa/L/s

Vitalograph Childrens

Respiratory Monitor

Accuracy: $\pm 5\%$ PEF

$\pm 3\%$ FEV_{1, -5, -75}

Resistance to flow: < 0.15 kPa/L/s

Review History

- 1 ⓘ Switch on



- 2 ⏪ Hold for 3 seconds



- 3 ▲ or ▼ Review history



- 4 ⏪ Return



Delete History

- 1  Switch on



- 2  +  Hold for 10 seconds



Cleaning and Disinfecting:

The outer surfaces should be cleaned every week, more often if necessary. The use of an ordinary alcohol wipe is recommended, with special attention to the mouthpiece area.

 www.vitalograph.co.uk 

Regulatory Notices

Warnings and Advisory Notices:

Note: Please read all this information before using this Vitalograph device. A full set of instructions, including cleaning instructions, is available at www.vitalograph.co.uk.

- This Vitalograph device is intended to measure lung function, primarily for single patient use at home under medical supervision or for use in clinic, where SafeTway or Bacterial Viral Filter disposable mouthpieces must be used.
- Take care not to block the mouthpiece with the tongue or teeth. A 'spitting' action or coughing will give false readings.
- If used at home symptoms must take precedence over device measurements*.
- If the device is used for longer than its specified life, the accuracy of the device may deteriorate.

- Before use, ensure that the batteries do not exceed their shelf life, as indicated on the batteries.
- Store in a clean dry place.
- Cleaning & disinfecting: The outer surfaces should be cleaned every week, more often if necessary. The use of an ordinary alcohol wipe is recommended, with special attention to the mouthpiece area.

** If the patient at home thinks that the device is not reading correctly, they must advise the healthcare professional immediately. Medical facilities may use a precision syringe to check the accuracy of this device.*

The device must be taken to separate collection at the product end-of-life.



Do not dispose of these products as unsorted municipal waste.

Warranty:


Your Vitalograph device is guaranteed for one year*. Replace if it is faulty, otherwise replace the unit every three years.

** Excepting accidental / transit damage or inappropriate use of the device.*


FDA Notice:

Caution: Federal Law restricts this device to sale by, or on the order of a physician.

CE Notice:

Marking by the symbol  indicates compliance of this device to the Medical Devices Directive of the European Community. Such marking is indicative that the Vitalograph device meets or exceeds the referenced technical standards. The Vitalograph device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any

interference in nearby electronic equipment. The Vitalograph device is battery operated and is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. The Vitalograph device is intended for use in the electromagnetic environment specified below. The customer or the user of the system should assure that it is used in such an environment.

- Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
- Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
- Interference may occur in the vicinity of equipment marked with the following symbol - .

Technical Specifications

Material: PC/ABS

Accuracy:

Better than $\pm 3\%$ (FEV₁ & FEV₆), $\pm 10\%$ (PEF)

Flow Impedance (as applicable):

Better than 0.15kPa/L/s at 14L/s

Measurement Range:

PEF: 25 - 840 L/min BTPS

FEV_{0.5}: 0 - 9.99 L BTPS

FEV_{0.75}: 0 - 9.99 L BTPS

FEV₁: 0 - 9.99 L BTPS

Safety & Performance Standards:

EN ISO 23747:2009, ISO 26782:2009, ATS/ERS
Guidelines 2005

Electromagnetic emissions:

CISPR 11 Group 1 (battery operated)

Electromagnetic immunity:

IEC 61000-4-2, IEC 61000-4-3 (battery
operated)

Sensor: Stator/rotor

Power Supply: AAA batteries

Operating Temperature: 17 – 37°C

Recommended separation distances between portable and mobile RF communication equipment and the system

The Vitalograph device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters and the system as recommended below, according to the maximum output power of the communications equipment.

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to the transmitter manufacturer.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80MHz to 800MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5GHz $d = 2.3\sqrt{P}$
0.01	0.1m	0.1m	0.2m
0.1	0.4m	0.4m	0.7m
1	1.2m	1.2m	2.3m
10	3.7m	3.7m	7.4m
100	11.7m	11.7m	23.3m

Note 1: at 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Declaration of Conformity

Product: Vitalograph Model 4000

Vitalograph hereby ensures and declares that the above product associated with this user manual, is designed and manufactured in accordance with the following QMS regulations and standards:

- European Medical Devices Directive {MDD} 93/42/EEC. This device, classified as 2a as per Annex IX of MDD 93/42/ EEC, meets the following provisions of Annex II of the Medical Devices Directive as per Article 11, section 3a, excluding point 4 of Annex II. This device complies with the EMC Directive 2004/108/EC, conformance demonstrated by following standard EN 60601-1-2:2007. Equipment classification: Residential.



- Canadian Medical Device Regulation {CMDR}.
- FDA Quality System Regulation {QSR} 21 CFR 820.
- EN ISO 13485: 2003. Medical devices. Quality management systems. Requirements for regulatory purposes.


Certifying Body {for 93/42/EEC and CMDR}:
British Standards Institute {BSI}.

Certificate Nos. CE 00772, MD 82182,
FM 83550.

Signed on behalf of
Vitalograph (Ireland) Ltd.

A handwritten signature in blue ink, appearing to read "B. R. Garbe". The signature is fluid and cursive, with the first letters of the first and last names being capitalized and prominent.

B. R. Garbe. Group Managing Director

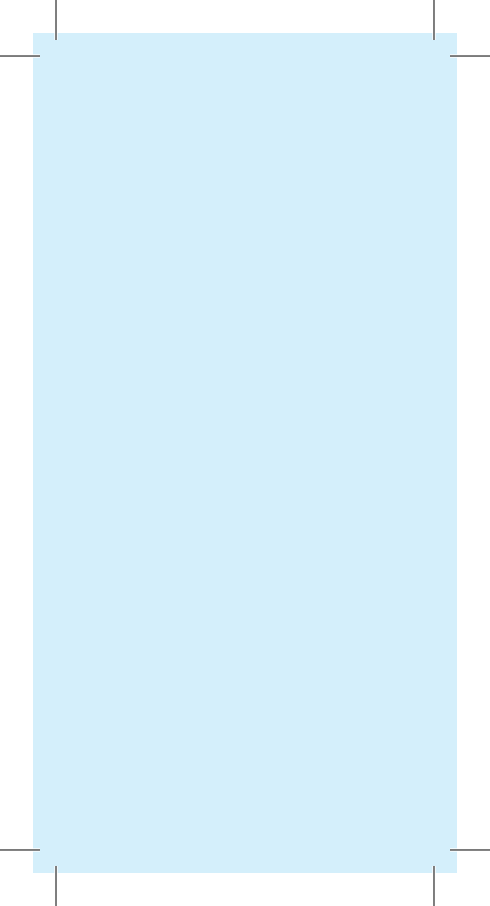

UK Sales Vitalograph Ltd.

Maids Moreton Buckingham MK18 1SW England

Tel: 01280 827110 Fax: 01280 823302

e-mail: sales@vitalograph.co.uk

www.vitalograph.co.uk

Export Sales Vitalograph Ltd.

Maids Moreton Buckingham MK18 1SW England

Tel: +44 1280 827120 Fax: +44 1280 823302

e-mail: sales@vitalograph.co.uk

www.vitalograph.eu

Vitalograph GmbH

Rellinger Straße 64a 20257 Hamburg Germany

Tel: (040) 547391-0 Fax: (040) 547391-40

e-mail: info@vitalograph.de

www.vitalograph.de

Vitalograph Inc.

13310 West 99th Street Lenexa Kansas 66215 USA

Toll Free: 800 2556626

Tel: (913) 730 3200 Fax: (913) 730 3232

e-mail: vitcs@vitalograph.com

www.vitalograph.com



Vitalograph (Ireland) Ltd.

Gort Road Business Park Ennis Co Clare Ireland

Tel: (065) 6864100 Fax: (065) 6829289

e-mail: sales@vitalograph.ie

www.vitalograph.ie

 **Vitalograph®**

Your respiratory partner

Vitalograph® and asma-1™ are trademarks or registered trademarks of Vitalograph Ltd.

PRINT REF: 07583_2


0086